## •• INDICATION

- Stopping bleeding and reducing inflammation in gum retraction, performing fingerprint procedures, restoring class V, and more.
- Reduction of excretion of sucral fluid during fixation of indirect adhesive restorations.
- The drug is intended for local application and can be used both with impregnated, and with impenetrable threads.
- It does not stain the sprained areas and tooth fabrics.

## •• COMPOSITION

- Aluminum Sulfate U.S.P. – 25.0%
- 8-Hydroxyquinoline – 0.1%
- Colored Gel – ad 100%

## •• INDICATIONS

For the purpose of aiding in gingival retraction and to control bleeding and reduce mucous membrane inflammation associated with dental impression procedures and Class V restorations or other tissue management problems in other phases of dentistry. A more recent use is to assure sulcular fluid control during luting of indirect bonded restorations. This product is for topical use only. Alu-Jen Gel is compatible with any type of impregnated or plain cord. Free from coloring the area prepared.

## •• DIRECTIONS FOR USE

Attach delivery tip to Alu-Jen Gel syringe. Apply a thin bead of gel directly into the sulcus area. Select cord and pack into gel. Wait 2 – 6 minutes. If still bleeding, rinse with air/ water spray and reapply gel. Remove cord. Rinse and dry. Take impression as usual.

## •• SPECIAL RECOMMENDATIONS

The use of double cord pack technique is recommended. Place a smaller cord first, followed by a larger size. The small cord can be left in while the impression is taken and then removed.

## •• CONTRAINDICATIONS

- None.

## •• WARRANTY

The manufacturer warrants the quality of manufactured products. The adverse events inflicted by violation of user manual, storage conditions and other events inflicted by non-stipulated usage of the material are not the subjects of warranty.

The customer is responsible for determination of suitability of this product for user's application. Warranty conditions: the product does not comply with requirements declared by manufacturer. In this case the manufacturer replaces the defective material within warranty period.

## •• LIMITATION OF LIABILITY

The manufacturer's liability is limited by only cases stipulated by direct legislation of the country.

## •• STORAGE

- Store at temperature of 4 – 25 °C. Protect from direct light. Close tightly the cap after use.
- Shelf life – 2 years.
- DO NOT FREEZE!

## •• RECYCLING

Dispose of the medical device in accordance with local / regional / national / international legal requirements.

## •• PACKING

- 5 ml in syringe delivery tips.

# JenD<sup>®</sup>

For professional use in dentistry only!

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Gel lubricant for lubrication of the tool during instrumental retraction of the gums when removing silicone prints

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