- Biflu-Jen is a quick drying dental varnish containing a micro-fine suspension of inorganic Fluorides.
- Its low viscosity allows it to be easily applied to form a thin films and so serve as an insulator of exposed root surfaces.
- The inorganic Fluorides in Biflu-Jen allow for deep fluoridation when applied to floor of cavities or as a form of prophylaxis against caries and a source of fluoride ions when applied topically to the teeth.
- Biflu-Jen also contains Thymol as a barrier against local bacterial activity.
- Biflu-Jen is transparent and therefore very convenient to apply for a dentist and as well for a patient as it does not cause the discoloration of the teeth and dries quickly.

•• INDICATIONS

- Caries prophylaxis.
- Fluoridation against caries.
- Management of exposed root surfaces.
- Protection of root caries lesion.
- Local reduction of bacterial activity.
- Topical application of fluoride following routine scale and polish.
- Post operative management of acid etched enamel surface.
- Insulation of tooth surface following mechanical treatment.
- Cavity liner, protective physical barrier.
- Dentinal tubule sealant.

•• DIRECTIONS FOR USE

- Surfaces to be treated are cleaned and thoroughly dried with air.
- Biflu-Jen must be shaken well before each use. Solid particles must be well dispersed.
- When applied to the tooth surfaces, simply drip one or two drops of Biflu-Jen onto a foam pellet, and smear a thin layer upon the tooth surface.
- Avoid forming thick film of varnish as this will tend easily to peel away.
- For lining of cavities drip one or two drops onto an application brush and then use it to smear a layer of varnish onto walls of the prepared tooth cavity.

- The varnish is allowed to be absorbed for several seconds and then dried with air.
- As a result Biflu-Jen forms the water impermeable film. With correct application and proper mouth hygiene the film remains in place for several days.

•• COMPOSITION

- Sodium Fluoride 0.75%
- Calcium fluoride 0.75%
- Thymol – 0.5%
- Natural resins in volatile solvent ad 98%.

•• SUPPLEMENTARY INDICATIONS

- The application of Biflu-Jen Improved can be repeated two or three times at intervals of 5 – 7 days.
- Surfaces should be dried and treated with Biflu-Jen again.

•• WARRANTY

The manufacturer warrants the quality of manufactured products. The adverse events inflicted by violation of user manual, storage conditions and other events inflicted by non-stipulated usage of the material are not the subjects of warranty. The customer is responsible for determination of suitability of this product for user's application. Warranty conditions: the product does not comply with requirements declared by manufacturer. In this case the manufacturer replaces the defective material within warranty period.

•• LIMITATION OF LIABILITY

The manufacturer's liability is limited by only cases stipulated by direct legislation of the country.

•• STORAGE

- Store at temperature of 4 – 25 °C. Protect from direct light. Close tightly the cap after use.
- Shelf life – 2 years.
- DO NOT FREEZE!

•• RECYCLING

Dispose of the medical device in accordance with local / regional / national / international legal requirements.

•• PRESENTATION

- Biflu-Jen – bottle 10 ml.

JenD[®]

For professional use in dentistry only!

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EN Biflu-Jen

Fluoride varnish & dentinal tube sealant

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