- Chela-Jen Gel is a gel designed to lubricate the instrument during instrumental treatment of root canals. The gel softens the calcium components of the walls of the canals and the dentin debris during machining process.
 - The EDTA contributes to the removal of calcium salts chelating the calcium in canal walls and makes it possible for the reamers and files to clean and shape the canals more rapidly. Its ability to clean deeply and to release dental tubules helps the penetration of the instruments into even the thinnest root canals. As an appropriate lubricant.
 - Chela-Jen Gel minimizes the risk of instrument breakage.
 - The chelating power of this substance enables the removal of calcium salts and pulp calculus with enhanced efficacy. Chela-Jen Gel also facilitates the withdrawal of fractured instruments and unfilling of old endodontic points (silver, gutta, acrylic, etc.).
- **INDICATIONS**
- Tool lubrication during root canal machining.
 - Chemical expansion of narrow and poorly accessible root canals with simultaneous disinfection.
 - Identification of channel mouths.
 - Unsealing previously sealed channels.
- **DIRECTIONS FOR USE (to be followed carefully):**
- After root canal irrigation with stabilized sodium hypochlorite solution introduce a file (K or H) coated with Chela-Jen Gel into the root canal while taking care of not reaching the apex. Chela-Jen Gel may be placed into canal with the delivery tips as well.
 - Finally the root canals should be treated with sodium hypochlorite and the H file.
 - Clean and dry up the root canals with drying and degreasing agent (i.e. Jen-CaviClean) and absorbent paper points. The filling of root canals will be achieved with an appropriate root-canal sealer (i.e. Endo-Jen, etc.) and points by using the routine procedure.

•• COMPOSITION

- EDTA 19%;
- Cetrimide 0.1%;
- Gel ad 100%.

•• CONTRAINDICATIONS

- Not found.

•• PRECAUTIONS

Avoid the contacts with skin and eyes. If contact occurs, flush well with copious amount of water. In case of ingestion – induce vomiting or seek medical assistance.

•• WARRANTY

The manufacturer warrants the quality of manufactured products. The adverse events inflicted by violation of user manual, storage conditions and other events inflicted by non-stipulated usage of the material are not the subjects of warranty. The customer is responsible for determination of suitability of this product for user's application. Warranty conditions: the product does not comply with requirements declared by manufacturer. In this case the manufacturer replaces the defective material within warranty period.

•• LIMITATION OF LIABILITY

The manufacturer's liability is limited by only cases stipulated by direct legislation of the country.

•• STORAGE AND SHELF LIFE

- Store at temperature of 4 – 25 °C. Protect from direct light. Close tightly the cap after use.
- Shelf life – 2 years.
- DO NOT FREEZE!

•• RECYCLING

Dispose of the medical device in accordance with local / regional / national / international legal requirements.

•• PACKING

- 5 ml syringe with delivery tips.

JenD[®]

For professional use in dentistry only!

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EN Chela-Jen Gel

Gel lubricant for lubrication of the tool during root canal tooling

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