## •• INDICATIONS

- For softening and desobturation of the root canals sealed with the phenol, formalin, and formaldehyde resins.

## •• DIRECTIONS FOR USE

Desobturation of the canals is performed by combined chemical action of Jen-Desobturat with mechanical reaming:

- Clean out the pulp chamber and canal orifice.
- Place a drop of Jen-Desobturat into the canal orifice.
- Desobturate the canal using a broach. After extraction out of the canal clean and put the tip of instrument into the Jen-Desobturat solution. Repeat the procedure step by step till the apex is reached.

## •• PRECAUTION

Avoid contact with eyes and minimize contact with oral soft tissue. If accidental contact with eye occurs, flush immediately with large amount of water.

## •• COMPOSITION

- DMFA – 98.6 g
- Excipient Q.S. – 100.0 g

## •• SUPPLEMENTARY INDICATIONS

Dry thoroughly the canal treated with Jen-Desobturat before filling by formalin/formaldehyde cement.

## •• WARRANTY

The manufacturer warrants the quality of manufactured products. The adverse events inflicted by violation of user manual, storage conditions and other events inflicted by non-stipulated usage of the material are not the subjects of warranty.

The customer is responsible for determination of suitability of this product for user's application. Warranty conditions: the product does not comply with requirements declared by manufacturer. In this case the manufacturer replaces the defective material within warranty period.

## •• LIMITATION OF LIABILITY

The manufacturer's liability is limited by only cases stipulated by direct legislation of the country.

## •• STORAGE

- Store at temperature of 4 – 25 °C. Protect from direct light. Close tightly the cap after use.
- Shelf life – 2 years.
- DO NOT FREEZE!

## •• RECYCLING

Dispose of the medical device in accordance with local / regional / national / international legal requirements.

## •• PACKING

- Bottle 9 ml.

# JenD<sup>®</sup>

For professional use in dentistry only!

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