Designed for cementing inlays and onlays as well as for other clinical situations where bonding to the etched enamel and dentin is involved. Jen-DuaCem is also recommended for fiber splints, screws, glass and carbon posts and veneers luting.

•• INDICATIONS

Jen-DuaCem is the restorative material intended for wide range clinical luting procedures especially where light curing cannot adequately reach. It can be used as:

- Indirect and nonlight cure bonding agent.
- The base for composite core fabrication.
- Chemically cure luting composite for conventional cast restorations.
- Luting material for bonded bridges (Maryland or Rochette type), screws, posts and splinting periodontically affected teeth.

•• DOSAGE AND MIXING

Jen-DuaCem in syringes 2 g (Part A and Part B).

- Place approximately equal amount of Jen-DuaCem component A and B on a mixing pad.
- Spatulate the two pastes together for approximately 20 seconds. Make certain that the blend is uniform.

Jen-DuaCem in double-barrel syringes 4.2 g or 8.4 g.

Before using for the first time, it is important to make sure that the material comes out evenly from both parts of the twin syringe. To do this, squeeze a small amount of material from the twin syringe onto the mixing pad to achieve even paste dosage and ensure appropriate mixing with the mixing pad later on.

Use a syringe with clean, disinfected gloves to avoid contamination. Do not reuse the syringe if it is contaminated. Disinfect the Jen-DuaCem twin syringe with a medium level disinfectant after each use.

- Turn the cap 90° anticlockwise, align the notch marks on the cap and the syringe and remove the cap.
- Attach the mixing nozzle, also aligning the notches on the nozzle and the syringe.
- Turn the mixing nozzle 90° clockwise and make sure it is correctly attached to the syringe before applying the material. The material should be squeezed out easily.
- Press gently on the piston to mix and apply the preparation. The two pastes are automatically mixed in their respective volumes. Additional manual mixing of the extruded material before use improves mixing quality!
- To remove the mixing nozzle once the material has been applied, turn it 90° anticlockwise, aligning the notches on the nozzle and syringe, and remove the nozzle. Fit the cap by aligning the notches and turn it 90° clockwise. Keep the two-cylinder syringe tightly closed.
- The mixing nozzle must be disposed of after use. Neither the empty syringe nor the used mixer may be reused.
- Clean used metal tools immediately after use with alcohol or acetone.

•• APPLICATION TECHNIQUE

- Good moisture control is important throughout the cementation procedure to prevent contamination of both the prepared tooth surface and the setting cement. This can be achieved through the use of a rubber dam or tissue retraction cord and cotton rolls with evacuation.
- Apply a thin layer of ready-mixed material to the restoration and the cavity, and place the restoration in the cavity. Avoid excessive filling of the cavity or restoration.
- Remove excess material with a suitable instrument. Avoid withdrawal the material from the gap. A small excess of material may be left in place, if required.
- Cure the exposed areas of material with dental light. Working with all-porcelain or composite restorative materials, cure Jen-DuaCem through the restoration for an adequate period of time.

- To prevent the formation of an oxygen inhibition layer, the barrier gel may be applied to the cemented fissures once the excess material is removed, but before the material is polymerized.

- If the material is allowed to harden with its self-cure mechanism only, the working time at 23 °C is approximately 120 seconds; it will fully cure in 6 to 8 minutes. When irradiated with dental curing light, the cement will achieve the initial hardness by irradiation for 20 – 30 seconds.

•• CLINICAL PROCEDURE FOR INLAYS AND ONLAYS PREPARATION

The «standard» inlay preparation should be rounded slightly and have butt margins. Feather edges or undercuts should be avoided. In onlays, 1,5 – 2 mm occlusal reduction is required.

Prior to Cementing. The «Total Etch» technique should be applied to prepared surfaces prior the sealing. All exposed dentin and enamel areas should be sealed with appropriate dentin sealer or adhesive (i.e. Jen-Unibond is recommended). If the ceramic restoration has not been silane treated at the lab, apply a Silane followed by Hydrofluoric Acid gel according to appropriate silanization technique. This will enhance the bond between the restoration and the cement.

Cementing. Apply a uniform layer of mixed cement to all surfaces to be bonded. Seat the restoration with a gentle pressure. Using a dental curing light irradiate the bonding area for 5 – 15 seconds while maintaining the pressure. The soft-cured material should facilitate to maintain the restoration in place, while allowing for easy removal of the excess. Hold restoration in place while trimming excess of cement. Use light unit or wait for 6 – 8 minutes to achieve the cement fully cured.

Finishing. After the cement is cured, trim around the margin, check occlusion, and adjust with a fine grit diamond. The bonded restoration should be polished at spots where adjustments have been made. Use rubber points followed by Diamond Polishing Paste.

•• WARRANTY

The manufacturer warrants the quality of manufactured products. The adverse events inflicted by violation of user manual, storage conditions and other events inflicted by non-stipulated usage of the material are not the subjects of warranty. The customer is responsible for determination of suitability of this product for user's application. Warranty conditions: the product does not comply with requirements declared by manufacturer. In this case the manufacturer replaces the defective material within warranty period.

•• LIMITATION OF LIABILITY

The manufacturer's liability is limited by only cases stipulated by direct legislation of the country.

•• STORAGE

- Store in dark place at 4 – 8 °C.
- Close tightly the cap after use.
- Shelf life – 2 years.
- DO NOT FREEZE!

•• RECYCLING

Dispose of the medical device in accordance with local / regional / national / international legal requirements.

•• PACKING

- Jen-DuaCem - 2 x 2 g syringes [Component A and Component B].
- Jen-DuaCem Set - 4 x 2 g syringes [Component A and Component B].
- Jen-DuaCem in double-barrel syringes 4.2 g or 8.4 g.

Shades: CHM (Chameleon), A2, A3, W0 (White Opaque).

JenD[®]

For professional use in dentistry only!

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