The material is intended for fixing pins, making stump inlays and restoring the stump of the coronal part. The specially treated fiberglass in the material not only provides high strength, but also ensures the required elasticity of the structure, which in turn helps to distribute the point load evenly over the entire volume of the structure. The material is characterised by high radiopacity and ease of use. Jen-DuaCem FR is supplied in a two-cylinder syringe with nozzles ensuring automatic and quality mixing of the material as well as high speed and accuracy of application.

#### •• INTENDED USE

Jen-DuaCem FR is intended for the following clinical procedures:

- Fixation of pins;
- Making of stump inlays;
- Restoration of the coronal stump;
- Used for both ceramic and composite restorations;
- Used as a restorative material for sandwich technique when coated with micro- or nano-hybrid composites.

#### •• TECHNICAL INFORMATION

- Polymerization mode: double polymerization.
- Working time: at least 90 seconds at a temperature of 23°C.
- Curing time: about 3-5 minutes at 37°C.
- Light curing: 20 seconds for a layer not more than 2 mm thick or 40 seconds for a layer not more than 4 mm thick.
- Please note that the data given are for a temperature of 23°C and a relative humidity of 50%. These values may vary depending on the conditions and period of storage of the material, temperature, air humidity. Increasing the temperature of the material reduces its operating time.

#### •• DIRECTIONS FOR USE

##### Dosage and mixing.

Before using for the first time, it is important to make sure that the material comes out evenly from both parts of the twin syringe. To do this, squeeze a small amount of material from the twin syringe onto the mixing pad to achieve even paste dosage and ensure appropriate mixing with the mixing pad later on.

Use a syringe with clean, disinfected gloves to avoid contamination. Do not reuse the syringe if it is contaminated. Disinfect the Jen-DuaCem FR twin syringe with a medium level disinfectant after each use.

- Turn the cap 90° anticlockwise, align the notch marks on the cap and the syringe and remove the cap.
- Attach the mixing nozzle, also aligning the notches on the nozzle and the syringe.
- Turn the mixing nozzle 90° clockwise and make sure it is correctly attached to the syringe before applying the material. The material should be squeezed out easily.
- Press gently on the piston to mix and apply the preparation. The two pastes are automatically mixed in their respective volumes. Additional manual mixing of the extruded material before use improves mixing quality!
- To remove the mixing nozzle once the material has been applied, turn it 90° anticlockwise, aligning the notches on the nozzle and syringe, and remove the nozzle. Fit the cap by aligning the notches and turn it 90° clockwise. Keep the two-cylinder syringe tightly closed.
- The mixing nozzle must be disposed of after use. Neither the empty syringe nor the used mixer may be reused.
- Clean used metal tools immediately after use with alcohol or acetone.

##### Fixation of pins.

1. Perform depulping and preparation of the space for inserting the pin, using standard techniques.
2. Apply etching gel (e.g. Phospho-Jen AS) according to the manufacturer's instructions. Rinse with distilled water for 20 seconds and dry the channel with a paper pin. Do not overdry.
3. Apply a layer of adhesive according to the manufacturer's instructions (use e.g. Jen-Unibond or Jen-Unibond SE). If the adhesive requires light curing, remove excess with a paper pin before curing.
4. Fill the channel with Jen-DuaCem FR material using channel filler. If desired, a thin layer of preparation can be applied to the pin. Insert the pin, allowing any excess material to come out freely and then remove the excess material.
5. Cure the material using a curing lamp where available or allow the material to cure chemically. In any case, the material requires about 5

minutes to cure sufficiently, especially in places that are difficult to reach for the light of a photopolymer lamp.

6. Cut the pin and restore the stump.

##### Making of stump inlays.

1. During restorative procedures, careful control of oral fluids is necessary to prevent contamination of both the prepared area and the cement. A good result can be achieved by using a cofferdam, gum retraction with threads and cotton rolls.
2. Prepare the dentin and enamel surface by applying an etching gel (e.g., Phospho-Jen AS) according to the manufacturer's instructions.
3. Rinse thoroughly with distilled water and dry the surface to remove excess moisture, but do not over-dry.
4. Apply a layer of adhesive over enamel and dentine according to the manufacturer's instructions (use Jen-Unibond or Jen-Unibond SE, for example).
5. Apply a second coat if the tooth is hypersensitive or is close to the pulp chamber. In the case of very deep prepared cavities, the use of calcium hydroxide spacer material is recommended. Remove any residual sealant or gasket from within the prepared cavity.
6. If using a matrix band, place the tip of the mixing nozzle directly on the prepared restoration site and squeeze out Jen-DuaCem FR. If using matrix moulds, fill the mould with material and place it on the prepared tooth. Wait approx. 5 minutes until the material is sufficiently chemically cured or light cure for 40 seconds on all sides.
7. After the cement has fully polymerised, grind the edges of the restoration or reshape the occlusal surfaces with fine diamond paste. Cemented restorations must be polished definitively in the areas where corrections have been made, using rubber polishing heads and diamond polishing paste.

#### •• PRECAUTIONS AND WARNINGS

- The material contains methacrylates, which may cause allergic reactions. Use medical gloves and avoid contact with skin, mucous membranes and eyes. In case of contact, rinse immediately with plenty of water and seek medical advice.
- The automatic mixing nozzles are exclusively disposable and must be disposed of after use.
- The use of materials containing eugenol may inhibit the polymerisation of the composite.
- Light curing is recommended in a thin layer using a halogen or LED lamp of 500 mW/cm<sup>2</sup> or more and a blue light emission spectrum of 400-500 nm.

#### •• WARRANTY

The manufacturer warrants the quality of manufactured products. The adverse events inflicted by violation of user manual, storage conditions and other events inflicted by non-stipulated usage of the material are not the subjects of warranty. The customer is responsible for determination of suitability of this product for user's application. Warranty conditions: the product does not comply with requirements declared by manufacturer. In this case the manufacturer replaces the defective material within warranty period.

#### •• LIMITATION OF LIABILITY

The manufacturer's liability is limited by only cases stipulated by direct legislation of the country.

#### •• STORAGE AND SHELF LIFE

- The material is used at room temperature and can be stored briefly at a temperature not exceeding 25°C.
- Close the cap tightly immediately after use.
- Long-term storage is recommended at 2 - 8°C to ensure maximum shelf life.
- Shelf life: 2 years.
- DO NOT FREEZE!

#### •• RECYCLING

Dispose of the medical device in accordance with local / regional / national / international legal requirements.

#### •• PACKING

Jen-DuaCem FR in twin syringes 4.2 g or 8.4 (component A, component B).

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EN Jen-DuaCem FR

Fiber-reinforced dual-cure composite  
cement

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