Jen-Esthe Post is made of clear quartz fibers pressure-bonded with a super strong epoxy resin. Jen-Esthe Post pins have elasticity, resiliency and thermal expansion indicators close to similar indicators of tooth dentin.

•• PURPOSE JEN-ESTHE POST

- Restoration and strengthening of the crown part of the tooth, including highly aesthetic restorations.

•• PROPERTIES JEN-ESTHE POST

The unique optical properties of the Jen-Esthe Post pins allow to strengthen restorations up to transparent enamel areas without harming the aesthetic component. Such a clinical decision makes the restoration more resistant to mechanical loads occurring in the oral cavity, and therefore increases its reliability and durability.

Post	Post size (inches = mm)	Reamer	Reamer size (inches = mm)
Jen-Esthe Post #1	0.039" = 0,9906	Jen-EndoDrill #1 (blue)	0.042" = 1,0668
Jen-Esthe Post #2	0.048" = 1,2192	Jen-EndoDrill #2 (green)	0.050" = 1,2700
Jen-Esthe Post #3	0.052" = 1,3208	Jen-EndoDrill #3 (yellow)	0.055" = 1,3970
Jen-Esthe Post #4	0.059" = 1,4986	Jen-EndoDrill #4 (black)	0.061" = 1,5494

•• REAMER JEN-ENDODRILL

- The J-EndoDrill reamer is also significantly different from the reamers we are used to. It is a miniature drill with a passive, non-cutting side surface. The J-EndoDrill reamer allows to make perfectly cylindrical preparation of the canal for accurate placement of the pin.
- The length of the working part of the tool is 16 mm.
- For ease of operation, three markings corresponding to distances of 14, 12 and 10 mm from the tip of the tool are applied to the surface of the reamer. These marks, which are clearly visible during the rotational movement of the tool, allow to control the depth of penetration of the reamer into the canal. For the dentist's convenience, reamers are also color-coded.
- The maximum speed of rotation is 500-1000 revolutions/min.

•• APPLICATION METHOD

WARNING: Jen-Esthe Post pins and cutters are supplied in non-sterile packaging and must be sterilized before use.

- Determine the pin size by comparing the reamer to the x-ray.
- Use a ball bur to remove intracanal material from the canal mouth and create guides (pilot hole) for the J-EndoDrill reamer.
- Set the rubber stopper on the reamer at the desired depth according to the X-ray image.
- If you are using pin analogues, install a rubber stopper or clamp the analogue with pliers in a place that corresponds to a certain reamer size.
- At the specified speed, install the tip of the reamer in the pilot hole. Easily, without effort, pass the intracanal material to the desired depth marked by the stopper. While continuing to hold the reamer at the specified revolutions, pull it out of the canal. In order not to damage the apical plug, do not stop the reamer either when inserting or withdrawing it from the canal.
- Determine the correctness of the pin log preparation by installing a pin identical to the reamed size.
- You can use pin analogues to check the prepared log. This eliminates the use of the main pin before planting. Use a rubber stopper or pliers to check the depth of the pin. Determining the correctness of the log preparation can be completed with an X-ray image, since the analogs of the pins are radiopaque.

- Cut the pin to the desired length using a high-speed diamond bur or disc. If you used a finishing pin to determine the quality of log preparation, you need to clean the pin with alcohol or chlorhexidine and dry it thoroughly.

NOTE: You can avoid the step of cleaning the pin if you use an analog to check the quality of the log preparation. The pin will remain clean until the moment of fixation.

- Etch the canal and cavity. Use a bonding system compatible with dual-cure cements. Follow the manufacturer's instructions. Jen-Unibond and Jen-DuaCem are recommended for use.
- Apply double-hardening cement into the canal. Fill the canal from bottom to top. Keep the nozzle embedded in the cement to avoid air bubbles.
- Immediately insert the pin into the canal. Excess cement can be squeezed out slightly and distributed around the pin, protecting it from excessive contamination.

- Position the light guide well above the tip of the pin. Polymerize for 40 seconds. The focusing nozzle on the light guide will help to focus the energy clearly on the pin.
- Immediately start restoring the tooth stump.

•• WARRANTY

The manufacturer warrants the quality of manufactured products. The adverse events inflicted by violation of user manual, storage conditions and other events inflicted by non-

stipulated usage of the material are not the subjects of warranty.

The customer is responsible for determination of suitability of this product for user's application. Warranty conditions: the product does not comply with requirements declared by manufacturer. In this case the manufacturer replaces the defective material within warranty period.

•• LIMITATION OF LIABILITY

The manufacturer's liability is limited by only cases stipulated by direct legislation of the country.

•• STORAGE AND SHELF LIFE

- There are no special storage restrictions.
- The shelf life is not affected.

•• RECYCLING

Dispose of the medical device in accordance with local / regional / national / international legal requirements.

•• PACKING

- Package: 6 pins of one specific size;
- Package: 1 reamer of a certain size;
- Set: Jen-Esthe Post Kit.

Composition of the set Jen-Esthe Post Kit:

- Jen-Esthe Post #1 – 6 pcs;
- Jen-Esthe Post #2 – 6 pcs;
- Jen-Esthe Post #3 – 6 pcs;
- Jen-EndoDrill #1 – 1 pcs;
- Jen-EndoDrill #2 – 1 pcs;
- Jen-EndoDrill #3 – 1 pcs.

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EN Jen-Esthe Post – fiberglass dental posts
Jen-EndoDrill – drills for posts placement

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