•• CAUTIONS

Jen-Favorite LC contains methacrylic resins that may cause an allergic reaction by skin contact in certain individuals. Avoid long or repeating contact of not-polymerized material with the skin [allergic contact dermatitis is possible], soft tissue of the oral cavity and eyes.

In case of contact, immediately wash thoroughly a place of contact with water and soap. If there is rash or other signs of allergic reactions on the skin stop use of the material and ask for medical care.

Use of this product in patients with known acrylic allergies is to be avoided.

•• PRECAUTIONARY MEASURES

- During the work with the material use special glasses, gloves, clothes and mask Goggles are recommended for the patients also.
- Observe the security measures provided during the work with powdery, siliceous substances. Use goggles, a mask and gloves.

•• COLLATERAL REACTIONS

The product can cause irritation of eyes, skin and mucous membranes. (see. CAUTION section).

•• INTERACTION WITH OTHER DENTAL MATERIALS

Use of eugenol-containing materials in a combination with Jen-Favorite LC is contraindicated. Eugenol-containing dental materials can have negative impact on ability to a polymerization.

•• WARRANTY

The manufacturer warrants the quality of manufactured products. The adverse events inflicted by violation of user manual, storage conditions and other events inflicted by non-stipulated usage of the material are not the subjects of warranty. The customer is responsible for determination of suitability of this product for user's application. Warranty conditions: the product does not comply with requirements declared by manufacturer. In this case the manufacturer replaces the defective material within warranty period.

•• LIMITATION OF LIABILITY

The manufacturer's liability is limited by only cases stipulated by direct legislation of the country.

•• STORAGE

- Inappropriate storage conditions will reduce shelf life and can lead to deterioration of properties of the material. Don't expose this product to direct sunlight. Store a material in a dry place.
- Store this product at the temperature of 4 - 25 °C [39,2 – 77 °F].
- DO NOT FREEZE!
- Shelf life of Jen-Favorite LC is 3 years.

•• RECYCLING

Dispose of the medical device in accordance with local / regional / national / international legal requirements.

•• PACKING

- Jen-Favorite LC in sets of 4, 6, 7 or 12 syringes.
- Jen-Favorite LC is supplied in syringes of 2 or 4 g.

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